

The Nu-Comfort Belt

Are you or your patients tired of a 1 inch wide belt that rides up or feels like a rope? If the answer is “yes” consider our softer, wider belt.

Twice the width of standard ostomy belts

Easy Velcro adjustment on left & right

360° of support to entire wafer


Extra support to the peristomal area

In addition, supports small peristomal hernias

Latex Free | Patent Pending



14
Openings
Available

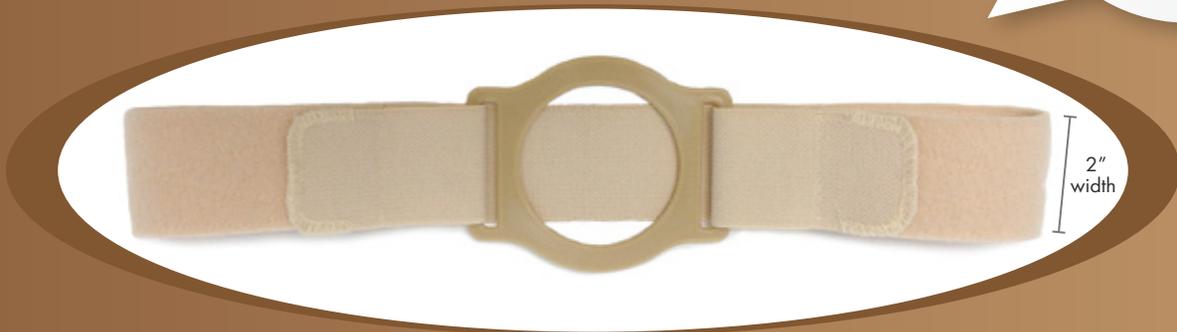

nu-hope.com



 /nuhopelabs

How to find the right size & generate a catalog number

Belt Elastic

#BG2620	SMALL	28" - 32" (71cm-79cm)
#BG2622	MEDIUM	32" - 36" (81cm-89cm)
#BG2624	LARGE	36" - 41" (91cm-102cm)
#BG2626	X-LARGE	41" - 47" (104cm-117cm)
#BG2628	XX-LARGE	47" - 52" (119cm-132cm)



Ring Plate

* Inner Diameter

F = 2 1/4"	A = 2 3/4"	C = 3 1/4"	* * Fits Nu-Hope Oval Pouches Only
G = 2 1/2"	DC = 2 7/8"	V = 3 3/8"	
S = 2 3/8"	B = 3"	H = Med Oval**	
I = 2 5/8"	U = 3 1/8"	J = Large Oval**	
W = 2 5/8" x 3 1/8"	Q = 2 7/8" x 3 3/8"		

The goal is to choose the closest-fitting ring plate, without pinching the sides of the pouch.



Fig.1

Fold the sides of the pouch inward as shown, take measurement across the center and + 1/8" (3mm) or 1/4" (6mm) depending on pouch.

Find the Ring Plate that matches your measurement and add its letter after the chosen belt elastic.

Example: #BG2622-B (2" Wide, Med, w/ 3" Ring Plate)

How to wear your Nu-Comfort Belt

1. Fold in pouch sides and slide pouch down through the Ring Plate.

2. The Ring Plate should be between the wafer and the pouch.

3. Slide the belt's Velcro (hook) strip through, from under the Ring Plate's belt slot, out and over so the Velcro (loop) fastens on the outside of the belt.

BELT CARE AND WASHING:

Press **hook & loop firmly together before laundering** Hand wash or machine wash warm with gentle cycle. Do not soak in hot water, wash with delicates, iron or use bleach. **Hang dry only.**

